



# Beta-Cath™ 3.5F System

VBT Designed for the Cath Lab

# Beta-Cath™ 3.5F System

- 6F guide catheter compatible
- Integrated markers for rapid sizing and positioning
- Beta radiation avoids non-target dosing
- Choose 30, 40 or 60 mm source for preferred margins
- Long-lived Strontium-90 for predictable dwell times

## Vascular Brachytherapy (VBT) Designed for the Cath Lab

### Best™ Radioisotope

- Strontium/Yttrium-90 pure beta emitting sealed source
- Optimum energy transfer from beta radiation
- Long radioactive half-life allows short, predictable treatment times
- 12-month service cycle
- Proven dose prescriptions and calculated dwell times provided with every device

### Best™ System

- The leader in vascular brachytherapy
- 6F guide catheter compatibility
- Portable/designed for the cath lab
- Indicator of Source Train (IST) wire aids in Jacketed Radiation Source Train (JRST) selection and simulates a “dummy run” of the JRST to ensure catheter lumen patency
- Multiple fixed length JRSTs available for optimal lesion coverage (30 mm, 40 mm and 60 mm)

### Best™ Procedure

- Short dwell time – keeps you on schedule
- Short treatment time – approximately 3 to 4 minutes
- No facility modifications, e.g. shielding, are required for any Cath Lab to use the system
- Passive centered 3.5F delivery catheter allows perfusion around the delivery catheter
- Dose profile allows clinicians to remain with patient

### Best™ Service

- Best Vascular provides all device service and radioactive source disposal
- Comprehensive on-site VBT education and in-service programs
- Long-life user-exchangeable battery powers device sensors, indicators and interlocks

### Safety Features

- Safety interlocks designed to ensure Jacketed Radiation Source Train containment
- Indicator of Source Train Wire allows rapid catheter positioning, injury sizing for train length selection and JRST lumen testing before irradiation
- Jacketed Source Train allows uniform unit-dosing with vessel conforming, and patented fast hydraulic movement
- All fluids contained in closed system
- Clinician whole body dose per treatment from VBT is less than 5% of the dose received from PTCA fluoroscopy


**Available in 30 mm,  
40 mm & 60 mm  
Source Train Lengths**

### Jacketed Radiation Source Train (JRST)

- JRST series of sealed miniaturized beta sources in a coil “jacket” forms a train, designed to provide even dose distribution
- JRST allows quick and easy position verification
- JRST is designed to maintain flexibility to navigate tortuosity while keeping all sources together
- Multiple fixed length JRSTs available for optimal lesion coverage (30 mm, 40 mm and 60 mm)



Detail of seeds inside the jacket

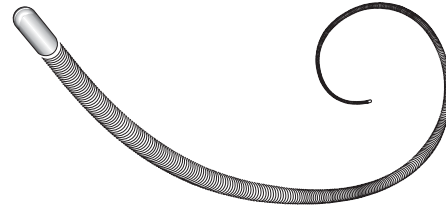

Jacketed Radiation Source Train (JRST)

### Exchangeable Battery

- An exchangeable battery powers the transfer device to allow for easy exchange of the product’s power source



### Transfer Device

- Portable and reusable
- Stores and delivers JRST
- Uniquely designed for the cath lab

### $\beta$ -Rail™ 3.5F Delivery Catheter

- Smallest delivery catheter available
- Easily fits in a 6F guide catheter and allows access to distal anatomy
- Single catheter accommodates any source train length
- 1 cm distal rapid exchange type catheter



### Patented Hydraulic Delivery

- Provides rapid source train movement
- No source handling or loading required

**Beta-Cath™ 3.5F System**  
Expanding the Capability

### Novoste™ Beta-Cath™ 3.5F System Intended Uses

The Beta-Cath™ 3.5F System is intended to deliver beta radiation to the site of successful Percutaneous Coronary Intervention (PCI) for the treatment of in-stent restenosis in native coronary arteries with discrete lesions (treatable with a 20 mm balloon for the 30 mm and 40 mm Systems and injury areas up to 40 mm for the 60 mm System) in a reference vessel diameter ranging from 2.7 mm to 4.0 mm.

### Contraindications

- Unprotected left main disease (>50% narrowing).
- Patients in whom antiplatelet and/or anticoagulant therapy are contraindicated.

### Warnings

- Every attempt should be made to avoid restenting of the target lesion to minimize the risk of thrombosis.

### Delivery Catheter & Source Train Placement

- Use of an Internal Mammary (IM) Artery Guide Catheter may impede the path of the ACTIVE Source Train and may cause unintentional exposure of radiation and/or unintended results.
- Vessel trauma may result from the improper use of the Delivery Catheter. Follow the enclosed directions carefully. When the Delivery Catheter is in the body, it should be manipulated only under fluoroscopy. Never advance or withdraw the Delivery Catheter against resistance without first determining the reason for the resistance under fluoroscopy.
- Failure to correctly position the Source Train at the injury site may underexpose the targeted treatment area and expose tissue not targeted for treatment to unintentional radiation. Exceeding the prescribed radiation treatment time will result in a higher than intended dose. Migration or improper location of the Source Train may cause unintentional radiation exposure to occur. The effect of unintentional radiation exposures and higher than intended doses are unknown.
- Do not over-tighten the hemostatic valve as this may damage the Delivery Catheter and impede the path of the Source Train and may cause unintentional exposure of radiation and/or unintended results.
- Upon return of the Radiation Source Train, failure to open the hemostatic valve may prevent the Radiation Source Train from returning to the device and may result in unnecessary radiation exposure to the patient or personnel.
- Failure to comply with the specific use of the Transfer Device controls may result in injury or unintended radiation exposure. Radiation is emitted from the Transfer Device when the Radiation Sources are in the Source Chamber. To minimize hand dose, the Transfer Device is designed to be held on the underside and may also be set down when appropriate.

### Intravascular Radiation Procedure

- If the fluid in the capped Fluid Collection Bag after the procedure is found to be contaminated after scanning, then the Transfer Device and capped Fluid Collection Bag should be placed in the Temporary Storage Container. Immediately inform Institutional Radiation Safety personnel, implement contamination control procedures and call your Best Representative.
- If, at any time, a Survey Meter reading of the Transfer Device, Delivery Catheter, Fluid Collection Bag, or Procedure Room is significantly different from initial baseline readings, stop all activity and re-survey the Transfer Device, Delivery Catheter, Fluid Collection Bag, or Procedure Room making sure the fluoroscopy is off. If the reading is not within the acceptable baseline range or background range there may potentially be a misplaced source; refer to the Emergency Source Recovery Procedure in the System Instructions for Use.
- **UNDER NO CIRCUMSTANCES** should an individual attempt to remove the Radiation Source Train from the Beta-Cath™ 3.5F System, grasp the catheter directly with hands, cut the catheter, or pick up a source with his/her fingers, because unintended radiation exposure and injury may result. Required equipment is provided for this purpose in the Response Kit.
- **Should breach of Source Train containment occur:**
  1. Notify personnel present of missing source(s).
  2. Follow institutional procedures regarding personnel allowed to enter or leave the room until the source is contained.
  3. Individuals involved in source recovery should wear

disposable gloves, an extremity dosimeter on the hand expected to receive the highest dose and a whole body dosimeter on the front of the body between the neck and the waist.

### Precautions

- The Beta-Cath™ 3.5F System is designed to be used by a team of appropriately trained personnel. At a minimum, this team should include a cardiologist, radiation oncologist, and medical physicist.

### Beta-Cath™ 3.5F System Preparation

- Prior to any procedure, the equipment should be thoroughly examined to verify the proper function and integrity of the system.
- Use the Delivery Catheter and Procedure Accessory Pack before the expiration date noted on the package. Verify that the sterility of the devices has not been compromised by assuring the package integrity has been maintained. The Delivery Catheter and Procedure Accessory Pack items are intended for single use. Do not re-sterilize and/or reuse these items.
- Do not use the Delivery Catheter if there is evidence of damage. If the Delivery Catheter Integrity Test detects a breach of catheter integrity, or restricted movement of the Source Train, note the observation and return the Source Train to the Transfer Device. Return the Delivery Catheter to Best Vascular, Inc. Prepare another Delivery Catheter for use.
- Handle the Transfer Device carefully and do not use if dropped. Do not use the Transfer Device if the controls and indicators are not functioning correctly or the LED light test is not observed. Do not begin a procedure if the Low Battery light is blinking. If the Low Battery Indicator starts blinking during a procedure, there will be enough battery power to complete the procedure.
- Do not use saline as a hydraulic fluid in the Transfer Device; corrosion may occur.
- The Transfer Device is not sterile. A sterile bag is provided to maintain a sterile field during the procedure. The inside portion of the tape covering the Syringe Port Hole and the Proprietary Connector Port Hole of the sterile bag is not sterile; remove from the sterile field.
- Use Caution when connecting the Proprietary Connector to the Transfer Device. The Proprietary Connector of the Delivery Catheter is no longer sterile once disconnected from the Transfer Device.
- Use care when attaching components to the Transfer Device to ensure that the Sterile Bag does not get pinched in the process.
- Ensure a sufficient number of sterile water-filled syringes are available before beginning treatment. Always reserve at least 10 ml of sterile water for the return of the Source Train to prevent unintentional radiation exposure.
- Ensure that the Gate Control Switch is completely closed, as incomplete closure may render the Gate inoperable.

### Intravascular Radiation Procedure

- Utilize a manual Blood Pressure Cuff to monitor patient status during the radiation treatment because arterial wave form pressure may be dampened while Delivery Catheter is in place.
- The Transfer Device contains radioactive material. Use of this device is restricted to persons licensed in the handling of radioactive materials. Personnel handling this device must follow the regulations, policies and procedures for their institution on the safety and hazards associated with radioactive materials.
- The individual performing the wipe and leak tests for radioactive material should use good contamination control techniques.
- If the transferable contamination exceeds 200 dpm/100 cm<sup>2</sup> (or the level determined by local regulation or institutional policy) or the leak test results exceed 11,100 dpm (or the level determined by local regulation or institutional policy) on any sample – place the contaminated object(s) in a plastic bag and label "Caution Radioactive Material." Immediately inform institutional Radiation Safety personnel, implement containment control procedures and call your Best Representative. Should this occur, do not continue with this procedure.
- Illumination of the Red Pressure Indicator light during a procedure indicates excessive pressure is being used – reduce applied pressure to return to the Amber Pressure Indicator area to avoid excess use of fluid.

- Do not turn the Transfer Device power On or attempt to open the Gate Control Switch during the Drying Procedure.
- Failure to perform adequate visual and radiation surveys post-procedure to verify source accountability may subject patients and/or personnel to unintended radiation exposure.
- The transfer device for the Beta-Cath™ 3.5F System has an exchangeable battery and the device requires scheduled maintenance by Best Vascular every 12 months.
- In the event a source becomes loose or needs to be transferred to a safe location, use the Source Recovery Tools with extreme care in source recovery. Improper use could damage sources and could potentially release unsealed radioactive material. Use of the Source Recovery Probe is the preferred method as it minimizes potential damage to the source.

### Special Considerations

- As with other vascular brachytherapy procedures, safety and effectiveness has not been demonstrated in the following populations:
  - Patients undergoing or having prior chest radiotherapy.
  - Patients unable to tolerate the recommended dwell time of the Source Train in the Delivery Catheter.
  - The safety and effectiveness of the Beta-Cath™ 3.5F System have not been evaluated in reference vessel diameters <2.7 mm.
  - Patients requiring revascularization methods other than balloon angioplasty, directional and rotational atherectomy and excimer laser for revascularization of in-stent restenosis.
  - Vessel or lesion morphologies that would preclude revascularization or placement of the Beta-Rail™ 3.5F Delivery Catheter.
  - Patients presenting with: thrombotic lesions, multiple vessel lesions, vein graft segments, overlapping stents, myocardial infarction less than or equal to 72 hours prior to the procedure and/or ejection fractions less than 30%.
  - Patient who have received a heart transplant.
  - Women of child-bearing potential who are pregnant or suspect pregnancy.

### Potential Adverse Events

The following adverse events were NOT observed during the clinical investigation, but are recognized as potential adverse events associated with the non-radioactive portion of vascular brachytherapy include (but not limited to):

- Arrhythmia • Arterial Damage, Dissection or Perforation • Vascular Access Site Hematoma • Contrast-Induced Nephrotoxicity • Neurologic Complications • Allergic Reactions • Infection • Stroke • Thrombotic Occlusion • Renal Insufficiency • Coronary Artery Bypass Graft Surgery • Slow Flow-Phenomenon • AV Fistula • Pseudoaneurysm • Left Ventricular Dysfunction • Systemic Atheroembolization • Endocarditis • Distal Embolizations • Vasospasm • Arterial Perforation • Retroperitoneal Hematoma

**Additional potential adverse events associated with the radiation portion of vascular brachytherapy include, but are not limited to:**

- Radiation Induced Malignancy • Aneurysm • Excessive radiation exposure to patient/staff • Arterial Damage • Coronary Artery By-pass Graft Surgery • Thrombosis • Restenosis • Myocardial Infarction • Death

*Best Vascular, Inc. is the exclusive owner, manufacturer and distributor of the Novoste™ Beta-Cath™ 3.5F System.*

*Novoste, Beta-Cath, β-Cath, β-Rail, the Beta-Cath System logo design and the Beta-Cath 3.5F System logo design are trademarks of Best Vascular, Inc.*

*U.S. Patent Numbers 5,683,345; 5,899,882; 6,013,020 and 6,261,219. Other patents pending.*

*© 2009–2018 Best Vascular, Inc. All rights reserved.*



Best Vascular, Inc. 4350 International Blvd Norcross, Georgia 30093 USA  
Tel: 770 717 0904 800 668 6783 www.novoste.com www.bestvascular.com

AFRICA | ASIA | EUROPE | LATIN AMERICA | MIDDLE EAST | NORTH AMERICA